SURGICAL CONTINENCE

# AJUST<sup>®</sup> HELICAL

ADJUSTABLE SINGLE-INCISION SLING

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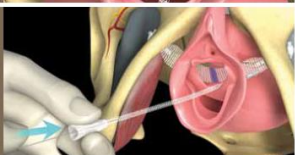
## Procedure Overview

<p><b>Step #1</b></p> <p>After appropriate dissection is completed, <b>push</b> the Fixed Anchor into the tissue until it is slightly beyond the ischiopubic ramus.</p>		<p><b>Step #6</b></p> <p>Stabilise the Adjustable Anchor at its insertion point while gently pulling on the Adjustment Tab.</p>	
<p><b>Step #2</b></p> <p>Rotate the handle (on its axis) driving the fixed anchor through the obturator internus muscle/membrane complex (typical rotation of the handle will be between 45 and 60 degrees, not to exceed 90 degrees). Ensure that the midline indicator is at or slightly past the mid-urethra in the direction of insertion.</p>		<p><b>Step #7</b></p> <p>Adjust the Sling.</p>	
<p><b>Step #3</b></p> <p>Release the Fixed Anchor by pushing the Anchor Release Lever forward and remove the Introducer. Apply gentle traction to the Sub-Urethral Sling to confirm proper fixation.</p>		<p><b>Step #8</b></p> <p>To loosen, apply gentle counter-traction to the Sub-Urethral Sling on the adjustable side of the Sling Implant.</p>	
<p><b>Step #4</b></p> <p>Load the Adjustable Anchor into the Introducer and secure by retracting the Anchor Release Lever.</p>		<p><b>Step #9</b></p> <p>Once proper Sling placement is achieved, insert the Flexible Stylet into the Adjusting Tab opening, and push the Sling Lock into place, up to the Adjustable Anchor.</p>	
<p><b>Step #5</b></p> <p>Repeat steps #1 - #3 on the contralateral side.</p>		<p><b>Step #10</b></p> <p>Withdraw the Stylet and trim the excess Adjustment Mesh lateral to the urethra at the level of the anterior sulcus. Close the vaginal incision using suture.</p>	

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**BARD Srl**